



There are two easy ways to measure your best tube length. Either method is sufficient.

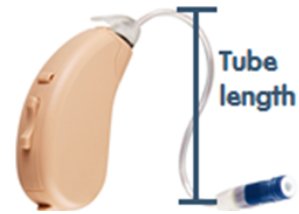


Method 1: Use a ruler

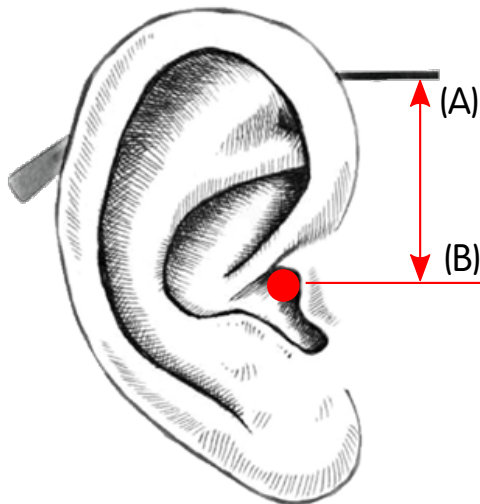

You can use a pair of glasses

Measure the distance from the underside of your glasses frame (A) to the top third of your ear canal (B)

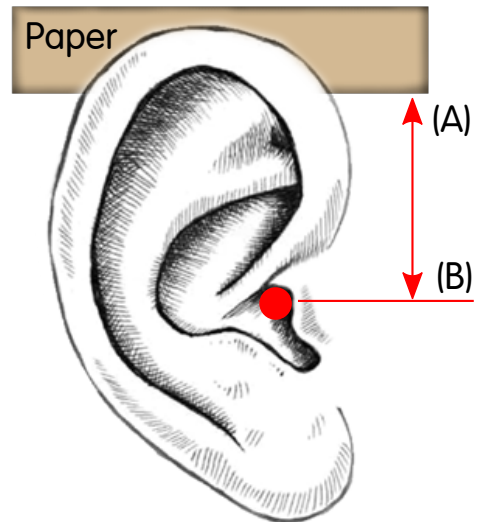

You can use a sheet of paper

Rest a piece of paper horizontally on top of your ear, and measure from the bottom of the paper (A) to the top third of your ear canal (B)

The table below indicates your tube length


Measurement:	Size:	SIE-64 Plus Opus-96	LOF
22mm (± 2 mm)		Size 0	Size 0
25mm (± 2 mm)		Size 1	Size 0
28mm (± 2 mm)		Size 2	Size 1
32mm (± 3 mm)		Size 3	Size 2
35mm (± 3 mm)		Size 4	Size 3

If you fit within two ranges, then pick the larger size. It is very uncommon for men to measure size 0; if you measure size 0, please double-check.

Blamey Saunders hears


Method 2: Use a template

Print on A4 paper and cut out the templates
(or call us to be posted a cardboard version)

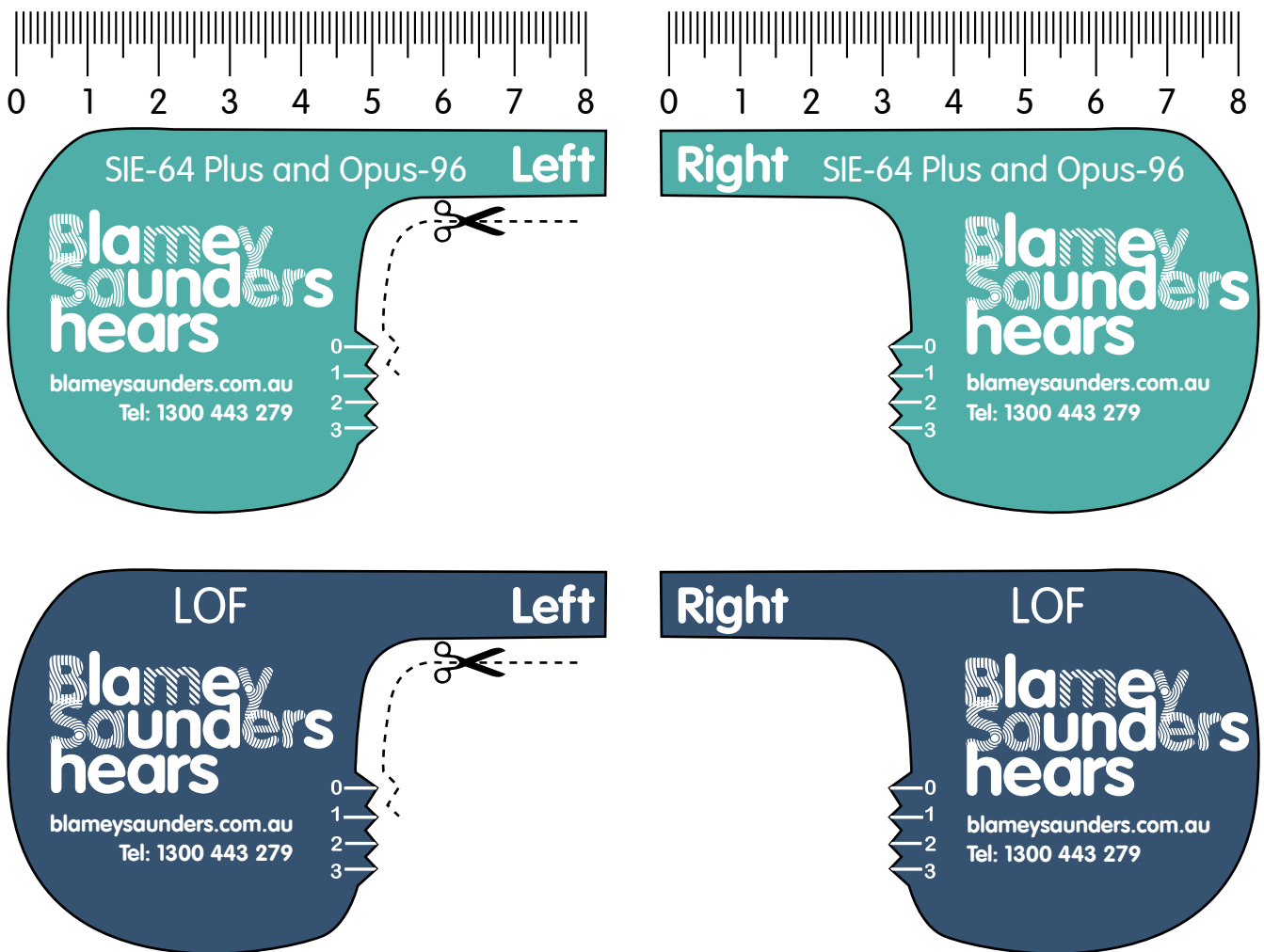

Step 1. Place the template on the ear as shown:

Step 2. Align the markers alongside your ear

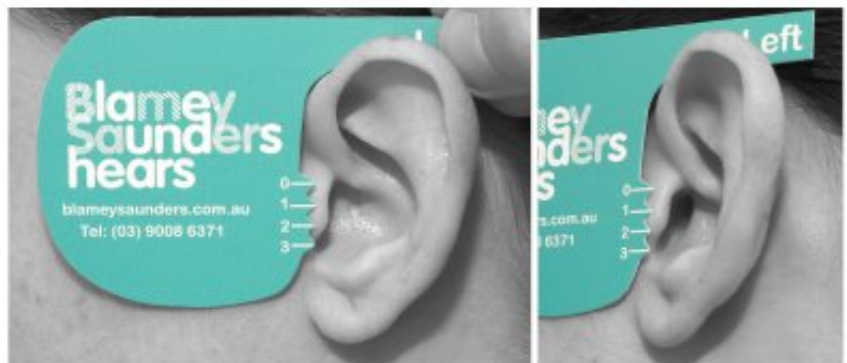

Step 3. Note which size aligns with your ear canal. This example shows a size 1 ear.